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zirconia primer

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Instruction for use

zirconia primer

1. Product description
Adhesive for zirconium dioxide frameworks. *zirconia primer* ensures a strong bonding with acrylates and zirconia ceramics and is designed for use in the dental laboratory for indirectly manufactured dentures.

2. Composition
Ethanol, MDP, Methacryloxypropyltrimethoxysilan

3. Intended Use
Saremc cements and auxiliary bonding promoters are intended for the indirect reconstruction or correction of functionally compromised natural dentition (e.g. deficient teeth).

4. Indication
Priming of frameworks of high-performance ceramics such as zirconium dioxide.

5. Contra-indication
Known allergy to methacrylates. *zirconia primer* is not suitable for conditioning composite frameworks.

6. Patient target group
zirconia primer is suitable for use in all patients without any age or gender restrictions.

7. User
zirconia primer should only be used by a professionally trained dental practitioner.

8. Side effects
In individual cases, contact allergies have been described to products of a similar composition.

9. Interactions
Avoid contact with materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, belong to this category.

10. Processing stages

10.1. Conditioning
Blast the zirconium dioxide frameworks/surfaces with 110 µ aluminium oxide at 2 - 3 bar (roughening, increasing the surface) and clean with oil-free compressed air. Then use a brush to apply the *zirconia primer* to the surface to be conditioned. Dry with oil-free compressed air.

Caution: If the air flow is too strong, the primer will be scattered, which will lead to a worse adhesion.

10.2. Further processing
Apply the veneering material/fixing cement to the surface to be treated and cure according to the manufacturers instructions. SAREMCO recommends els cem. Using dual-curing materials, additional light-curing is recommended (also through the respective restoration).

11. Storage
Protect light-curing products from strong sources of light and heat! *zirconia primer* was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C / 82°F can reduce the shelf-life of the product.

12. Batch number and expiry date
The batch number should be specified to identify products in case of enquiries. Products should no longer be used once the expiry date has elapsed.

13. Precautionary measures
Close containers after each use with the right lid. Keep out of reach of children. For dental use only. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes in contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

Hazard warnings: May cause an allergic skin reaction. Causes serious eye damage. Harmful to aquatic organisms, with long-term effect.

14. Emergency measures
In case of contact with the eyes, rinse cautiously with water for a few minutes. Remove any contact lenses, if possible. Continue rinsing. Consult an eye specialist.

15. Hygiene
Use application instruments for one patient only. Dose products away from patients to avoid contamination. The usual hygiene rules must be observed. Dispose of the content/ container in accordance with local, regional, national, and international regulations.

16. Warranty
The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user

EN | English

to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental.

17. Other notes for Europe
If the user and/or patient becomes aware of serious incidents in connection with the use of the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides. The Summary of Safety and Clinical Performance of SAREMCO Products can be found in the European database on medical devices (EUDAMED–<https://ec.europa.eu/tools/eudamed>).

18. Production / distribution
SAREMCO Dental AG
Gewerbestrasse 4
CH-9445 Rebstein / Switzerland
Tel: +41 (0) 71 775 80 90
Fax: +41 (0) 71 775 80 99
info@saremc.ch
www.saremc.ch

Edited 07-2025 | D600234

Class IIa medical devices
Class III medical devices (Canada)

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