BOHRINSEL

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BOHRINSEL

INSTRUCTIONS FOR USE

Attention!
Prior to first use, please check the scope of delivery for completeness and integrity!

START-UP


Do not carry using the laser arm!

Hold underneath the magnetic plate for lifting and carrying!

Electric connection, incl. mains plug and mains switch for switching the pin drilling unit ON and OFF.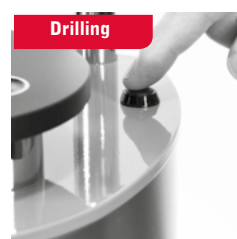



Release button for drilling process.

SETTING DRILLING DEPTHS

The drilling depth is preset ex works. The drilling Depth is adjusted with the Allen key under the drill in the drilling axis with the aid of the provided mounting tool.
Clockwise rotation:
Drill is lowered = narrower drill hole
Anti-clockwise rotation:
Drill is raised = wider drill hole

CHANGING THE DRILL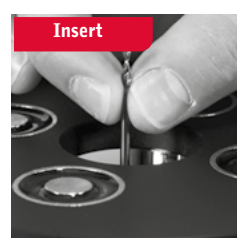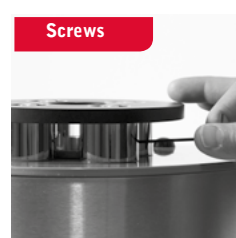

Always switch the unit off first and disconnect the mains plug. Loosen the two lateral grub screws in the drill spindle with the provided Allen key and pull out the drill. Insert the new drill and tighten both grub screws tightly and evenly. To set a new drilling depth, please proceed as described in section "Setting drilling depths". Switch the unit back on and perform test drilling.

Technical data

Dimensions D/W/H: 190 x 380 x 270 mm
Weight: 6.5kg
Electrical connection: 230V / 38W
Speed: 3.600 1/min
Laser class: 3A / 2.5 - 4.5mW

Scope of delivery

Bohrinsel pin drilling unit
Universal plate holder, 1 pc.
Pin drill conical, 1 pc.
Impression carrier L
Mounting tool

Warnings and safety notes

The drill must never be changed when the unit is switched on, please switch the unit off and disconnect the mains plug. If the unit is switched on, never place fingers or hands near the laser point. **Attention!** There is the risk of injury.

The Bohrsinsel pin drilling unit is not intended for the following purposes:

- In explosive environments
- For medical applications
- For processing wet materials

Klasse 4 Dental will not accept any liability if the Bohrsinsel pin drilling unit is not used in accordance with the instructions for use. Please observe the relevant regulations of the trade association when using the pin drilling unit:

- Use protective glasses
- Ensure adequate lighting
- Use a suction device

Attention: Repairs or other procedures may only be performed by Klasse 4 Dental or specialist personnel authorised by Klasse 4 Dental!

General notes & notes on care

Prior to operating the unit please check whether the power supply corresponds to the data on the type plate. Clean the pin drilling unit occasionally with a soft cloth. Do not use compressed air or unsuitable cleaning agents.

In case of any faults please send the device directly to Klasse 4 Dental or a specialist dealer authorised by Klasse 4 Dental. In-house repairs are performed exclusively with original spare parts.

Legal notices

All information in this documentation was prepared with the utmost care, changes and errors reserved. No legal responsibility for consequences due to erroneous data or inappropriate use.

Klasse 4 Dental GmbH

Bismarckstr. 21
D-86159 Augsburg
Phone +49 (0) 821 6089140
www.klasse4.de